MEDELA FLUID MANAGEMENT SYSTEM
THE INNOVATIVE SOLUTION FOR THE FUTURE

GEBRAUCHSANWEISUNG
INSTRUCTIONS FOR USE
MODE D'EMPLOI
ISTRUZIONI PER L'USO
GEBRUIKSAANWIJZING

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Medela's aim is to improve the quality of people's lives by means of innovative products

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The Basic 30 is a high-quality secretion aspirator. Its well-proven piston/cylinder system guarantees maximum suction performance and quiet, dependable operation. Additional advantages of the Medela secretion aspirators are: user-friendliness, simple cleaning and safety features. A comprehensive range of accessories makes the Basic 30 ideally suited to a wide range of medical applications. Contact us - we will be pleased to advise you!

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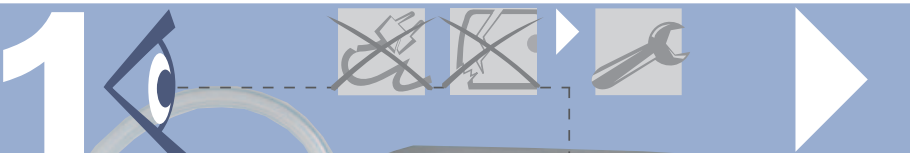
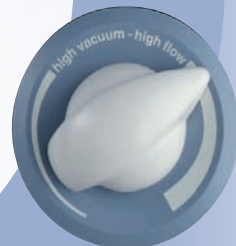
Before use, always check for correct operation!

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Maximum vacuum according to technical specifications

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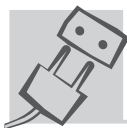
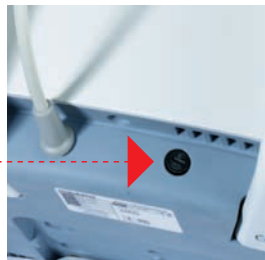
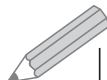
<i>Usage</i>	The appliance may only be operated by persons qualified in medical applications and familiar with in-house hygiene guidelines.
<i>Mains voltage</i>	Separation by unplugging the power cord.
<i>Overflow</i>	Contact your service center immediately. Work in accordance with the Service Manual.
<i>Routine inspection</i>	According to the Service Manual – once a year.
<i>Disposal</i>	The unit is made of metals and plastics and may only be disposed of in accordance with local regulations. The unit must be disabled before disposal.
<i>Warranty</i>	5 year warranty from the date of delivery for the unit. Medela is not liable for modified equipment, for any damage or consequential damage caused by incorrect operation, inappropriate usage as well as use by unauthorized persons. Routine inspections and servicing work must only be carried out by personnel authorized by Medela.

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Medical Electrical Equipment,
Model Basic 30 - Version 100-120 V/60 Hz
CLASSIFIED WITH RESPECT TO ELECTRIC SHOCK, FIRE,
MECHANICAL AND OTHER SPECIFIED HAZARDS ONLY
IN ACCORDANCE WITH UL 2601-1 AND CAN/CSA C22.2 NO. 601.1, 22P

Technische Änderungen vorbehalten
Subject to change
Sous réserve de modifications techniques
Con riserva di modifiche tecniche
Technische wijzigingen voorbehouden

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Medela Medizintechnik GmbH
Medela Benelux B.V.
Kuhn und Bieri AG