
BOY®S Kompressor


Type 053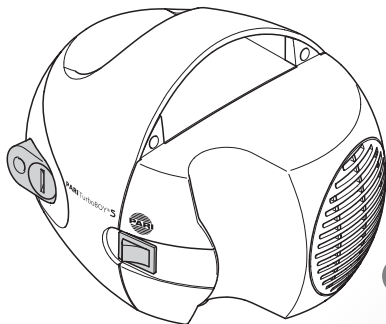

CE 0123







BOY®SX Kompressor






Type 085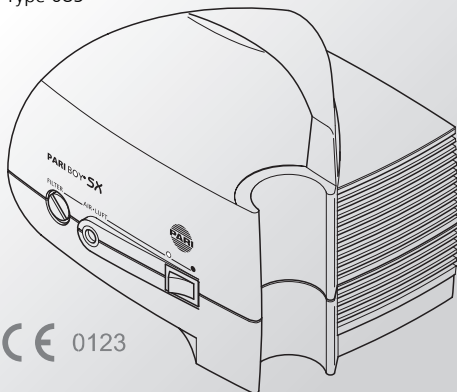

CE 0123

DE	Gebrauchsanweisung.....	1
EN	Instructions for use	15
DA	Brugsanvisning.....	27
EL	Οδηγίες χρήσεως.....	39
ES	Instrucciones de uso	51
FI	Käyttöohje	63
FR	Notice d'utilisation	75
IT	Istruzioni per l'uso	87
NL	Gebruiksaanwijzing	99
NO	Bruksanvisning	111
PT	Manual de Instruções	123
SV	Bruksanvisning	135
TR	Kullanım kılavuzu.....	147

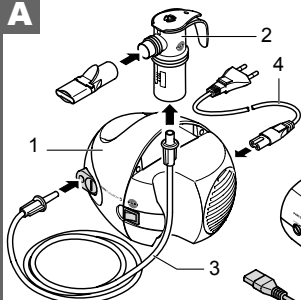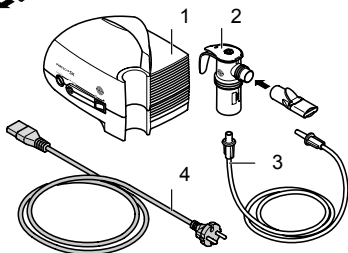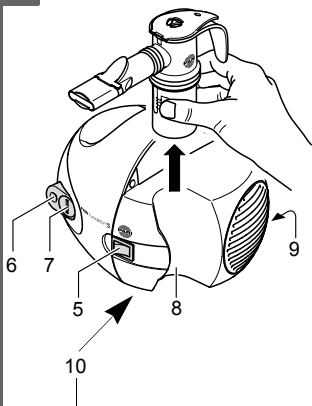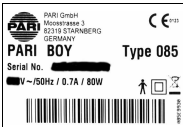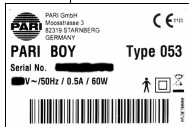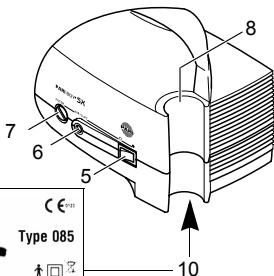A**PARI BOY S****PARI BOY SX****B****PARI BOY S****PARI BOY SX**

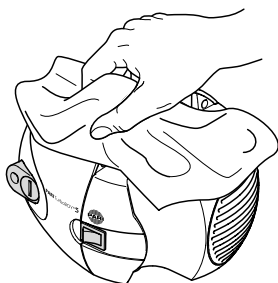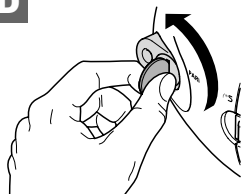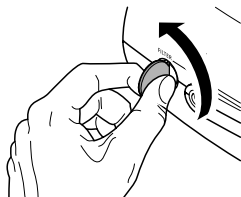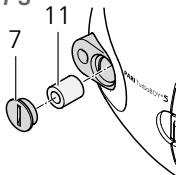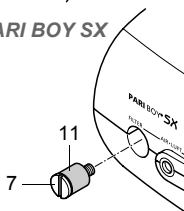C**D****PARI BOY S****PARI BOY SX**

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Stand der Information: September 2007

EN Instructions for use

Read these instructions carefully and completely before using for the first time. Keep these instructions in a safe place.

Risks caused by failure to comply with the instructions:

Failure to comply with the instructions may cause personal injury and/or damage the product. Specifically, failure to follow the instructions may have the following dangerous consequences:

- Failure of important functions of the product – resulting in inefficient treatment
- Danger of personal injury from heat, electrical, mechanical or other effects – in all variations and with all associated consequences

Table of contents

1	Components.....	16
2	Working parts.....	16
3	Important notes.....	17
4	Intended use.....	18
5	Preparation for use.....	19
6	Cleaning.....	20
7	Filter change	21
8	Disposal.....	21
9	Technical data.....	22
10	Shipping and storage	23
11	Terms and conditions of warranty	24
12	Explanation of symbols	25

1 Components

Please check that your PARI BOY S / PARI BOY SX compressor contains all the following components (see figure A):

PARI BOY S	PARI BOY SX
(1) PARI BOY S compressor	(1) PARI BOY SX compressor
(2) PARI LC SPRINT nebuliser ^{a)}	(2) PARI LC SPRINT nebuliser ^{a)}
(3) Connection tubing	(3) Connection tubing
(4) Power cord	(4) Power cord

a) For a list of components included in the accompanying nebuliser kit, see the instructions.

2 Working parts

The PARI BOY S / PARI BOY SX compressor includes the following working parts (see figure B):

(5) On/off switch	(9) Power cord socket (rear of appliance)
(6) Air outlet	
(7) Filter holder	(10) Label (bottom of appliance)
(8) Holder for nebuliser	

3 Important notes

The PARI BOY S / PARI BOY SX compressor is an electrically operated appliance. It must not be left unattended while it is in operation, or used in damp areas or where there is a risk of explosion.

Whenever the appliance is used, a thorough knowledge of the instructions for use and compliance with the same is assumed. In all cases, the operator is responsible for the safe operation of the appliance if it is not used appropriately for its intended purpose.

The appliance may only be serviced by PARI GmbH or a service center expressly authorised by PARI GmbH. If repairs are carried out by an authorised PARI GmbH service center, you must ask for a certificate giving the date, type and scope of the repair, and the company name and signature. Only original PARI accessories may be used to operate the appliance.

Please remove the power plug from the socket to ensure that the power is completely cut off.

Safety instructions:

- If the device or the power cord is damaged, or if a fault is suspected after the device has been dropped or suffered a similar shock, the system must not be operated and/or the power plug must be disconnected immediately.
- The power plug must be disconnected
 - if a malfunction occurs during operation
 - before cleaning and maintaining the appliance
 - after each use
- Do not pull the power plug out of the socket with wet hands; never pull on the power cord to disconnect the plug.
- Do not run the power cord over sharp edges or allow it to be pinched; do not let it hang unsupported and do not expose it to excessive heat or oil.
- Babies, infants, children, and anyone who requires assistance must be supervised constantly by an adult during inhalation therapy. Otherwise, safe and effective treatment cannot be guaranteed.
- Anyone standing below the compressor may be injured if it falls.

If PARI inhalers are rented or re-used, please follow the "Guidelines for preparing PARI inhalation devices for re-use on a change in patients" issued by the manufacturer. These guidelines are available upon request from PARI GmbH, or on the Internet at www.pari.de.

PARI GmbH is not responsible for damage or malfunctions caused by the operator using the machine improperly or contrary to the instructions.

4 Intended use

When used in conjunction with a PARI nebuliser, the PARI BOY S / PARI BOY SX compressor forms a highly effective inhalation device for treatment of the upper and lower respiratory tracts with nebulised medications. It is suitable for temporary¹⁾ oral or nasal inhalation of liquid medication prescribed or recommended by a doctor for the inhalation treatment of the lower and upper respiratory tracts.

The PARI BOY S / PARI BOY SX compressor must only be used with PARI nebulisers (except the PARI LC PLUS UNIVERSAL) and the correct PARI accessories. Using nebulisers and accessories made by other manufacturers can cause the inhalation device to be used improperly, which will impair the efficiency of the treatment because the PARI compressor and the PARI nebuliser are designed to complement each other.

You must always have a medical examination before treating illnesses. Only use medication prescribed or recommended by your doctor.

1) Each application lasts less than 1 hour, in this case max. 20 minutes.

5 Preparation for use

- The device should be at room temperature when it is used.
- Place the PARI BOY S / PARI BOY SX compressor on a flat, clean and dust-free surface.
- Do not run the compressor on carpets or in very dusty environments, e.g., under the bed, in workshops etc.
- Ensure that the side ventilation slits are not blocked.
- The compressor must not be covered under any circumstances during operation.
- Check that the supply voltage complies with the data marked on the appliance.

Figures **A** and **B**:

- Insert the connection tubing (3) with a slight twist into the air outlet (6) on the compressor.
Attach the other end to the PARI nebuliser (2).
- Connect the compressor (1) to the mains power supply:
 - Plug the power cord (4) into the socket (9) on the compressor.
 - Insert the mains plug of the power cord into a suitable socket.

To disconnect the PARI compressor from the mains after the treatment session, follow the above steps in reverse order, pulling the plug out of the socket **first**.

Please read the instructions for use of the nebuliser carefully and thoroughly and use the nebuliser accordingly.

- Before treatment, pull the nebuliser upwards out of the nebuliser holder (8).
- Turn the compressor on at the On/Off switch (5).
- After treatment, put the nebuliser back in its holder.

In order to avoid any risk to health (e.g., an infection), please also read the "Hygienic re-use" chapter in the instructions for use for the PARI nebuliser you are using. It is also important that you observe the subsequent instructions on "Cleaning" and "Filter change" in these instructions.

6 Cleaning

Before cleaning, turn the compressor off and remove the power cord from the socket.

Figure C:

- Wipe the outer surface of your PARI BOY S / PARI BOY SX compressor housing with a damp cloth.

Do not spray liquids into the ventilation slits. Such liquid may damage the electrical equipment and other compressor parts and cause a malfunction.

If liquid gets into the compressor, please contact an authorised dealer or the PARI Service Center (Tel.: +49-(0)8151-279 220 or info@pari.de). Do not use your PARI BOY S / PARI BOY SX compressor except on their instructions or after a check and any necessary repairs have been carried out by your authorised dealer or the technical Service Dept. of PARI GmbH.

7 Filter change

The filter on the front plate of your PARI BOY S / PARI BOY SX compressor must be replaced every 200 operating hours in normal operating conditions, or at least once a year.

Also please check the filter at regular intervals (after every 10 to 12 inhalations). If it is soiled (grey or brown colour) or blocked, replace it. If the filter has become damp, it must also be replaced with a new filter¹⁾.

Do not try to clean the filter and use it again.

Only use original PARI filters; otherwise your PARI compressor may be damaged or treatment may be inadequate.

To change the filter, proceed as follows (see figure D):

- Unscrew the filter holder (7) from your PARI BOY S / PARI BOY SX counter-clockwise using a suitable coin.
- Pull the old filter (11) out of the filter holder and fit the new filter in its place.
- Insert the filter holder with the new filter in your PARI BOY S / PARI BOY SX and screw it in firmly, turning clockwise with a coin.

8 Disposal

This product falls within the scope of the European Council Directive on Waste Electrical and Electronic Equipment (WEEE²⁾) and is included in product category 8: Medical devices for the purposes thereof. Accordingly, this product must not be disposed of with domestic waste. The disposal regulations prevailing in the respective member countries must be observed (e.g., disposal by local authorities or dealers).

Materials recycling helps to reduce the consumption of raw materials and protect the environment.

1) Part. no. 041B4852, available from your dealer or PARI GmbH.

2) Directive 2002/96/EC of the EUROPEAN PARLIAMENT AND THE EUROPEAN COUNCIL of January 27, 2003 on waste electrical and electronic equipment.

9 Technical data

PARI BOY S

Electrical connection:	see label on the bottom of the compressor
Maximum A-evaluated sound pressure level of the compressor:	approx. 56dBA (acc. to: EN 13544-1, section 26)
Housing dimensions:	(W x H x D) 18cm x 11.5cm x 16cm
Weight:	1.25kg
Pressure:	approx. 1.2bar
Flow:	approx. 4.5l/min

PARI BOY SX

Electrical connection:	see label on the bottom of the compressor
Maximum A-evaluated sound pressure level of the compressor:	approx. 56dBA (acc. to: EN 13544-1, section 26)
Housing dimensions:	(W x H x D) 19.2cm x 14.5cm x 15cm
Weight:	1.7kg
Pressure:	approx. 1.6bar
Flow:	approx. 5.1l/min

Classification according to EN 60601-1:1996 for PARI BOY S / PARI BOY SX


Type of electric shock protection:	Protection class II
Degree of protection from electric shock of the applied part:	Type B
Degree of protection against water penetration in accordance with IEC 529 (IP rating):	No protection
Degree of protection when used in the presence of flammable mixtures of anaesthetics with air or with oxygen or nitrous oxide:	No protection
Operating mode:	Continuous operation

Information concerning electromagnetic compatibility

- Portable and mobile high-frequency communication devices such as mobile phones can disrupt electrical medical equipment.
- The electrical field generated when the compressor is running can interfere with screens and monitors (flickering). If you notice any signs of interference (such as flickering) while the device is in use, move it farther away from other electrical equipment.
- Technical electromagnetic compatibility data is available in table form upon request from PARI GmbH, or on the Internet at www.pari.de, on the "Products/Inhalers/Technical data" tab.

10 Shipping and storage

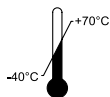Ambient temperature -40°C to $+70^{\circ}\text{C}$

RH
10% - 95%

Relative humidity 10% to 95%

P
500hPa-1060hPa

Atmospheric pressure 500hPa - 1060hPa

11 Terms and conditions of warranty

During the warranty period, we or our customer service will repair any defects in the compressor resulting from faults in material or workmanship free of charge. The owner shall not be entitled to cancel the sale or demand a partial or full refund of the purchase price. This warranty does not cover damage caused by improper use of the appliance. The warranty shall become void if repairs are carried out on the appliance by unauthorised persons. No compensation will be paid for consequential or direct damage.

In the event of a claim under this warranty, please bring the complete appliance to your dealer or send it to us, packed in the original carton and postage paid, together with this warranty certificate.

Wearing parts are excluded from the warranty.

The warranty period shall commence on the date of purchase.

12 Explanation of symbols

	- Caution, observe the instructions for use - Warning
	On/Off
	Alternating current
	Protection class of the part used: Type B
	Protection class II appliance
	The medical device was distributed commercially after August 13, 2005. This product must not be disposed of with normal domestic waste. The symbol of the refuse bin with a line through it indicates that it must be collected separately.

We reserve the right to make technical changes.

For all product information please contact our Service Center:

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