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Instruction manual

UK

Carefully read the complete user manual before using the toilet safety rail RFM for the first time. Keep the instruction manual safely, so that it is available for later use.

Intended use

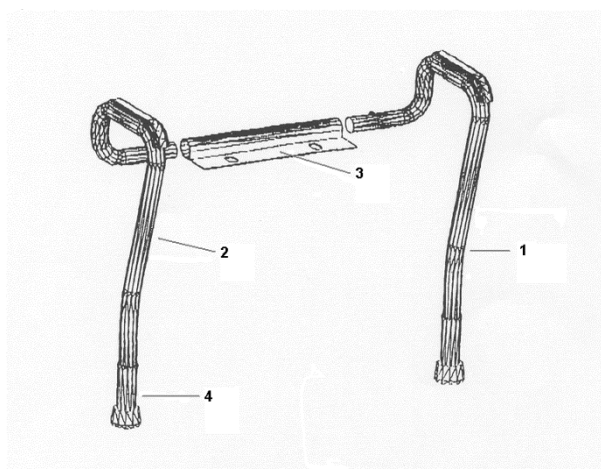
The toilet safety rail RFM supports people who are restricted in their mobility in the trunk and back, when sitting down on the WC seat and then standing up again.

indication

Suitable for patients, who have a limited ability to move, rotate and bend the trunk due to loss of strength or require support in standing up due to an accident.

Instructions / Mounting: Your authorised dealer should mount it or help you in mounting

1. Check the carton and the contents for damage and completeness. If it is damaged during transport, do not assemble.
2. Fix the connecting piece / bracket (3) to the toilet bowl with the screws that you also use for the toilet seat. Your dealer / your nursing staff will help you with this.
3. Slide the right and the left railing pipe (1+2) into the bracket, until they are locked audibly in place. You can select two widths.
4. Adjust the legs by means of the push buttons (4) to the same height, so that they are standing straight on the floor.

Safety Instructions

1. Check the tightness of the side rails before EVERY use!
2. Ensure that the connecting piece is fitted tightly to the toilet seat!
3. Make sure that both legs are placed firmly on the ground and are set at the same height.
4. Do not pull yourself up by the railing and do not pull the railing pipes.
5. Do not pull the railing pipes apart! The toilet safety rail RFM provides support in sitting down and standing up. If you suffer from a loss of strength, do not use the toilet safety rail RFM.
6. Do not brace yourself against the railing from the outside!
7. Do not hang any objects on the railing!
8. Do not leave children unattended in the bathroom / WC. The toilet safety rail RFM is not a gym apparatus or a play device!
9. Do not use a defective toilet safety rail RFM!

Cleaning

You should clean and regularly disinfect the toilet safety rail RFM after every use. This is compulsory if the chair is used and re-used by another person or patient! Clean it with a mild soap solution. Dry the toilet safety rail RFM with a soft cloth and make sure the bathroom is well ventilated.

The shower bench can be disinfected with a commercially available product according to manufacturer's instructions, eg. B. Descsept AF. The toilet safety rail RFM consists of materials that are protected against corrosion and are resistant to commercially available disinfectants and cleaning agents. Do not use abrasive or corrosive cleaning agents or high-pressure and steam cleaners!

Contraindications

The use of the toilet safety rail RFM is not indicated for people with balance disturbances and disorders of perception, severe visual impairment and paralysis that make the movement of the torso and back area almost impossible. In case of loss of the upper limbs, the use of the toilet safety rail RFM is not suitable.

Maintenance

The toilet safety rail RFM is maintenance-free. If you notice any damage or changes to the material or instability, please immediately contact your dealer!

Do not use damaged or defective toilet safety rail RFM!

Storage

Ensure that the bathroom is well ventilated during use, in order to prevent corrosion. Make sure the toilet safety rails RFM is cleaned and dried before storing it, when it is not being used. Then place the toilet safety rails RFM in the supplied plastic bag and box in a dry and dark place.

Specifications / Dimensions and weight

Height:	67-77 cm (incl. armrests)
Width:	54 - 62 cm
Weight:	approx. 1.5 kg
Max. load bearing capacity:	113 kg
Material:	anodised aluminium (not susceptible to corrosion)

Disposal

A disused toilet safety rail RFM is a valuable source of materials that can be supplied to the industrial cycle. Therefore, please do not throw the toilet safety rail RF in the household waste, but rather dispose it of at a state-approved disposal site. Your local waste collection services would be pleased to provide you with all the relevant information.

Warranty

The warranty period is 24 months and shall apply to all product defects that can be attributed to manufacturing or material defects.

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