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■ Mode d'emploi

Kuh
er
Bieri
Kuh
Bieri

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■ Rehausse WC souple

11 cm

Please read these notes carefully before use. Make sure to follow the instructions and keep them in a safe place ! If this aid is to be used by other persons, they must be fully informed of these safety warnings before use.

1. Safety indications

- before every use, check to make sure that the toilet raiser is in perfect condition
- the indicated maximum load of 185 kg must not be exceeded
- check regularly the stability of the toilet seat raiser
- for reasons of hygiene, the use of the toilet seat raiser must be restricted to one person only

2. Installation

Raise the toilet seat, insert the soft toilet raiser Contact, and check that the fixing lips extend beyond the pan, inside and outside. Check the stability of the toilet raiser before sitting. Once in place, it raises the toilet pan of 11 cm.

3. Cleaning

The toilet seat raiser has to be cleaned daily with a non-abrasive cleaner. It can be cleaned with products containing no chlorine, such as Surfanios (Anios laboratories), or with soapy water. Do not use any corrosive or abrasive detergents, or pressure cleaners. Sterilisation is possible at 90°C (194°F) and 80 % humidity for 5 minutes.

4. Materials

The toilet seat raiser is made of one piece. The skin is made of plastic, and the inside of foam. It raises the toilet pan of 11 cm.

5. Disposal

Dispose of this product in accordance with your regional waste disposal regulations.

6. Technical specifications

Maximal weight (user): 185 kg

*This product has a 2 years warranty, from the date of purchase, against any manufacturing defect.
This product is biocompatible.*

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Kuhn und Bieri AG