EN: Instructions for Use | **AR:** تعليمات الاستعمال | **BG:** Инструкции за употреба | **CS:** Návod k použití
DA: Brugsanvisning | **DE:** Gebrauchsanweisung | **EL:** Οδηγίες χρήσης | **ES:** Instrucciones de uso | **FI:** Käyttöohjeilla
FR: Notice d'utilisation | **HR:** Upute za uporabu | **HU:** Használati útmutató | **IT:** Istruzioni per l'uso | **JA:** 使用説明
KO: 사용 지침 | **NL:** Gebruiksaanwijzing | **NO:** Bruksanvisning | **PL:** Instrukcja użytkowania | **PT:** Instruções de
utilização | **SK:** Návod na použitie | **SL:** Navodila za uporabo | **SR:** Uputstvo za upotrebu | **SV:** Bruksanvisning
ZH: 使用说明

EN: Post-Surgical Bras | **AR:** حمالات الصدر المخصصة للارتداء بعد العمليات الجراحية
BG: Следоперативен сутиен | **CS:** Pooperační podprsenky | **DA:** Post-operative bh'er | **DE:** Postoperative BHs | **EL:** Μετεχειρηρτικοί στηθόδεσμοι | **ES:** Sujetadores postquirúrgicos | **FI:** Rintaliivit
leikkauksen jälkeiseen käyttöön | **FR:** Soutiens-gorge post-chirurgicaux | **HR:** Postoperativni grudnja-
ci | **HU:** Műtét utáni melltartók | **IT:** Reggiseni post-chirurgici | **JA:** 術後用ブラジャー | **KO:** 수술후 보정용 브
라 | **NL:** Postoperatieve beha's | **NO:** Postoperative BHer | **PL:** Biustonosze pooperacyjne | **PT:** Sutiãs
pós-operatórios | **SK:** Pooperačné podprsenky | **SL:** Modrčki za uporabo po kirurškem posegu | **SR:** Post-hirurški
grudnjaci | **SV:** Postoperativa bröstställare | **ZH:** 术后文胸

EN: The device in primary packaging is prepared for direct clinical use | **AR:** مداخلتس لآ لآ نم قيساسا ملوب ع يف زاولا الذ زيهت جنبي | **BG:** Устройството в първична опаковка е готово за директна клинична употреба | **CS:** Zdravotnický prostředek v
primárním balení je připraven k primárnímu klinickému použití | **DA:** Produkt i primær emballage forberedt til direkte medicinsk brug | **DE:** Produkte in einer Primär-
verpackung sind gebrauchsfertig und können direkt für eine klinische Nutzung verwendet werden | **EL:** Η συσκευή στην αρχική συσκευασία
παρασκευάζεται για άμεση κλινική χρήση | **ES:** El producto en su envase primario está destinado al uso clínico directo | **HR:** Proizvod u
primarnom pakiranju pripremljen je za izravnu kliničku uporabu | **HU:** Az elsődleges csomagolású eszköz közvetlen klinikai felhasználásra készül | **FI:** Kuluttajapakkaus-
sessa oleva tuote on valmis suoraan kliniseen käyttöön | **FR:** Dans son emballage primaire, le dispositif est préparé pour une utilisation clinique directe | **IT:** Il presidio
presente nella confezione è adatto all'uso clinico diretto | **JA:** 本製品は直接包装されており、臨床にもご使用いただけます。 | **KO:** 기본 포장
재의 기기는 직접 임상적 사용을 위해 준비되어 있습니다 | **NL:** Het hulpmiddel in primaire verpakking wordt voorbereid voor direct klinisch gebruik
NO: Enheten i primærpakningen er klargjort for direkte klinisk bruk | **PL:** Wyrób w opakowaniu podstawowym jest przygotowany do bezpośredniego użytku klinicznego
PT: O dispositivo na embalagem primária está preparado para utilização clínica direta | **SK:** Pomôcka v spotrebiteľskom obale je pripravená na priame klinické použitie
SL: Naprava v osnovni embalaži je pripravljena na neposredno klinično uporabo | **SR:** Uređaj u primarnom pakovanju pripremljen je za direktnu kliničku upotrebu
SV: Produkten i primärförpackningen är förberedd för direkt klinisk användning | **ZH:** 初级包装中的本品可直接用于临床。

3211 Post-Op Bra, Bianca | 3212 Post-Op Bra, Bella | 3270 Post-Op Bra, Anna | 3291 Post-Op Bra, Alice | 3342 Post-Op Bra, Sophia | 3343 Post-Op
Bra, Mary | 3373 Post-Op Bra, Marianne | 3444 Post-Op Bra, Ava | 3471 Post-Op Bra, Lisa | 3609 Post-Op Bra, Bonnie | 3831 Post-Op Bra, Bree
3954 Post-Op Bra, Alma | 3086 Comfort Bra, Catia | 5006 Comfort Bra, Celia 5007 Comfort Bra, Celia

EN	4
AR	6
BG	8
CS	10
DA	12
DE	14
EL	16
ES	18
FI	20
FR	22
HR	24
HU	26
IT	28
JA	30
KO	32
NL	34
NO	36
PL	38
PT	40
SK	42
SL	44
SR	46
SV	48
ZH	50
T 1-3 & 5	52-57

Intended purpose (Post-Op Bras): Fixate and support the breasts and protect wound and scar following breast surgery. Fixate bandages or other medical devices following breast surgery.

Intended purpose (Comfort Bras): Fixate and support the breasts and protect wound and scar when full movement of arm is regained following breast surgery and in radiotherapy period.

Users: Women, following breast surgery as described under Indications.

Contraindications

Relative contraindications:

- Very sensitive scar and skin tissue in the operated area
- Skin diseases
- Radiodermatitis
- Contact dermatitis

Indications

For use 24 hours per day following the surgeries in **T 1**

 Use the devices according to medical indication to avoid injury or damage.

Clinical benefits

- Fixation of bandages and wound dressings
- Protection of wound/scar
- Support and compression of the operated breast tissue

Compression Level

The Post-Surgical Bra is ranked according to each other in five levels of compression as experienced based on the compression and support zones. High (1), Firm (2), Intermediate (3), Moderate (4) and Light (5). Figure shows a compression level of 2 (Firm) and is displayed on the packaging for the specific bra and in **T 2**

Compression and support zones (Tytex ZoneTech®)

Table 4 – Firmness indicates the different knitted zones within the specific Post-Surgical Bra. The lower the number, the firmer the knitting; increased compression and support. Compression and support zones for each specific bra are displayed in **T 3**

ZONE	FIRMNESS
1	1
2	2
3	3
4	4
5	5
6	6

T 4 – Firmness

- The knitting is unique to each specific Post-Surgical Bra and cannot be compared across the bra range.

Care instructions

- Find care instructions in the bra
- Close the bra before washing
- Wash with similar colours
- Wash in a laundry bag
- Use of chlorine based products may decrease durability and function

Reporting obligation

If you have any complaints, please contact your supplier directly. In the events of a serious incident that could lead to a significant deterioration of health or to death, contact the manufacturer and the competent authority of the member state. Serious incidents are defined in article 2 No. 65 of Regulation (EU) 2017/745 (MDR).

Lifetime of the device in use

The lifetime of the medical device is determined by natural wear and tear, even when treated correctly and as recommended. In case of signs of deterioration affecting performance and comfort, the device should be disposed of.

Warnings and precautions

- Follow the Instructions for Use
- Follow your surgeon's recommendations regarding use and fitting
- If any unexpected discomfort or wound irritation occurs, remove the Post-Surgical Bra and contact your surgeon
- The Post-Surgical Bra is not intended for direct contact with injured skin or mucous membrane
- Make sure that bandages, textile breast prosthesis and/or drainage tubes are placed correctly to avoid pressure points and/or obstructed flow
- Before X-ray examination remove Post-Surgical Bra except; 3211 Post-Op Bra, Bianca, 3086 Comfort Bra, Celia and 5006/5007 Comfort Bra, Celia
- Wearing an undersized Post-Surgical Bra may cause local pressure pain and disruption of blood flow and/or lymphatic flow
- Wearing an oversized Post-Surgical Bra may reduce the clinical effect

Performance characteristics

General performance characteristics

-  Compression and support zones; Integrated and targeted compression and support zones.
-  4-way stretch: The bra adjusts to the operated breast along with the patient's changing breast contour after surgery.
-  Seamless: No seams in critical areas (cups, sides, back and under bust).
-  Flexible sizing: Bra fits several sizes due to 4-way stretch and size concept.
-  Breathable: Knitted material is breathable.

Specific performance characteristics **T 2**

Material composition

Raw materials and components used in the devices are tested for harmful substances according to STANDARD 100 by OEKO-TEX®, product class II, and relevant Biological Evaluation (ISO 10993). The device and packaging comply with REACH.

 Products may contain traces of softening and antistatic agents

T 5

PRODUCT	MAIN FABRIC COMPOSITION
3211 Post-Op Bra, Bianca	77% CO, 13% PA, 10% EL
3373 Post-Op Bra, Marianne 3609 Post-Op Bra, Bonnie 3212 Post-Op Bra, Bella 3291 Post-Op Bra, Alice 3270 Post-Op Bra, Anna 3954 Post-Op Bra, Alma 3086 Comfort Bra, Celia	91% PA, 9% EL
3342 Post-Op Bra, Sophia	78% CO, 15% PA, 7% EL
3343 Post-Op Bra, Mary	S-2XL: 91% PA, 9% EL 3XL-4XL: 94% PA, 6% EL
3831 Post-Op Bra, Bree 3444 Post-Op Bra, Ava	89% PA, 11% EL
3471 Post-Op Bra, Lisa	53% PA, 38% CO, 9% EL
5006 Comfort Bra, Celia 5007 Comfort Bra, Celia	60% CO, 25% PA, 15% EL


Damaged packaging

If the packaging is damaged, do not use the product. E.g. if the packaging is visibly damaged and/or the product itself is visibly compromised or contaminated, the product must be discarded.

Measuring bra size

1. Use a standard measuring tape
2. Stand up while measuring
3. Bust size ▲: Measure around the upper body, at the fullest part of the breast
4. Band size ■: Measure around the upper body, just under your breast
5. The difference between the bust size (▲) and the band size (■) shows your cup size (●)
(▲ - ■ = ●)
6. Find your bra size* in the size chart printed on the packaging** by using the band size ■ and the cup size ●

- ! * Due to possible post surgical swelling and edema, you may need to go 1-2 sizes up compared to your normal bra size. Consult your surgeon for guidance.
- ! ** The size guide printed on the packaging is unique to each specific Post-Surgical Bra and cannot be used across the bra range.

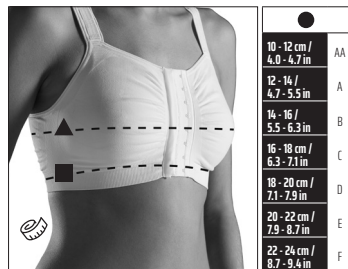Instructions for use

Depending on medical indication, Post-Op Bras can be combined with 8113 Implant Stabilizer band. 8108 Breast Puffs can be used with pocketed Post-Op Bras.

Fitting the bra and accessories

Post-Op Bras

STEPS	PRODUCT(S)	INSTRUCTION
0.	3954 Post-Op, Alma	Easily switch between straight or crossed straps before application by detaching the straps at the front, threading through the opposite top eyelet, and securing with the hook & loop closure.
1.	All Post-Op Bras	If medical condition allows, insert one arm and pull the bra over the shoulder. Then insert the other arm. If necessary, ask clinical staff for assistance.
2a.	3373 Post-Op Bra, Marianne 3609 Post-Op Bra, Bonnie 3212 Post-Op Bra, Bella 3291 Post-Op Bra, Alice 3343 Post-Op Bra, Mary 3831 Post-Op Bra, Bree	Pull the under bust band together. Fasten the bottom hook first and work up to the top. Use the widest setting that fits immediately following surgery and the narrowest setting when swelling has subsided.
2b.	3211 Post-Op Bra, Bianca	Pull the under bust band together. Work from the bottom to the top, pressing firmly on hook & loop to close.
2c.	3342 Post-Op Bra, Sophia 3471 Post-Op Bra, Lisa 3444 Post-Op Bra, Ava 3270 Post-Op Bra, Anna 3954 Post-Op Bra, Alma	Pull the bra together. Fasten the hooks first, followed by the zipper.
3.	All Post-Op Bras	Position each breast correctly in the cup. Avoid pulling the bra down too low.
4.	All Post-Op Bras	If the bra has adjustable shoulder straps, adjust these so the bra does not irritate the skin, provides support under the breasts and sides and feels comfortable to wear.

Comfort Bras

STEPS	PRODUCT(S)	INSTRUCTION
1.	3086 Comfort Bra, Catia 5006 Comfort Bra, Celia 5007 Comfort Bra, Celia	Pull bra over your head or pull it up like underwear.
2.	3086 Comfort Bra, Catia 5006 Comfort Bra, Celia 5007 Comfort Bra, Celia	Adjust the bra and ensure nothing is folded.

Textile Breast Prosthesis

STEPS	PRODUCT(S)	INSTRUCTION
1.	3373 Post-Op Bra, Marianne 3342 Post-Op Bra, Sophia 3343 Post-Op Bra, Mary 3471 Post-Op Bra, Lisa 3444 Post-Op Bra, Ava	Remove fiberfill from the device until size fits.
2.		Place the device in the pocket before bra is fitted.

Mounting drainage bulb

STEPS	PRODUCT(S)	INSTRUCTION
1.	3373 Post-Op Bra, Marianne	Place drainage bulb (max 500 ml) in strap and place strap on bra. Press firmly on hook & loop.

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Carefix is a registered trademark of Tytex A/S © 2024.
All rights reserved Tytex A/S, Marsvej 6, DK-7430 Ikast
Tytex A/S has been certified according to ISO 9001, ISO 14001 and ISO 13485
TYTEX A/S • Marsvej 6 • DK-7430 Ikast • www.tytex.com