Smart**Watcher**[®]

SPIRIT

UPDATES TO THE USER GUIDE | 2021.03.03

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TO POWER OFF THE WATCH

Step 1

Press the lower button ③ twice quickly.

Step 2

When you see the “Power off” watch face, please select:

- a. the **green icon (right)** ✓ to confirm you want to Power off the watch
- b. the **red icon (left)** ✗ to cancel the Power off

To go back to a previous watch function

simply click once on the lower button ③

TEST CELLULAR NETWORK CONNECTIVITY

EN

IMPORTANT!

To support the performance of your watch, please **test the connectivity at home and any areas where you will regularly wear the watch**. While the SPIRIT shows excellent cellular network connectivity, low network connectivity in general can impact the watch's ability to send an alarm.

Please note that if there is no cellular connectivity on the watch, a call cannot be established.

NOTE

With SPIRIT specifically, the watch works very hard to connect. When cellular connectivity in the area is low, it may become warm and give itself an automatic cooling period. When cooling, the watch shows that it is in airplane mode and cannot connect a call. After cooling, the watch will return to the watch face visual or give you the option to press “reload.”

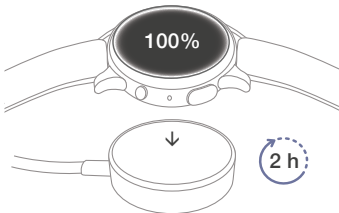
We recommend that you test your watch alarm periodically (i.e. once a month) to ensure it is connecting properly.

BATTERY LIFE AND WATCH USE

EN

Plan a daily battery charging routine that works for you. Personal alarm calls will impact the battery level. We suggest that you find a time to charge your watch that fits your daily rhythm so that the watch is charged when you most want to wear it.

If you have any further questions, please contact us at
support@smartwatcher.com
or
call us directly:

International:	+41 43 508 46 92
For Switzerland:	+41 43 508 46 92
For Austria:	+43 800 010 067
For Belgium:	+32 2 234 37 37
For The Netherlands:	+31 085 888 6516
For France:	+49 1573 599 1600
For Italy:	+39 02 8295 6070

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© 2021 Smartwatcher Technologies AG
Smartwatcher Technologies AG