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DE

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## Bathtub board with handle

EN

Read the complete operating instructions carefully before using the urine bottle holder Standard for the first time. Keep the instructions for use so that they are available for later use.

### Intended use:

The bathtub board with handle makes it possible to take a shower at the level of the bathtub edge. It makes it easier for persons with restricted mobility to clean the body and to dry it while sitting.

### Indication:

For persons with limited mobility because of paralysis or joint diseases, the bathtub board serves as a support when getting in and out of the bathtub. Taking a shower in a sitting position is suitable for people who are not able to stand during body cleansing.

### Instructions:

1. Remove all the parts of the bathtub board from the carton and check for completeness and damage.

2. To mount the four rubber holders, insert a plastic screw from the top through the opening / groove and tighten the rubber holder below the board. Using the enclosed washers, the brackets can be adjusted to the desired width in the groove. The screws must be held securely in place.

3. The handle is mounted on one side of the board. For this purpose, the handle is inserted from above into the two round openings of the bathtub board and fastened with two screws to the bathtub board from the bottom.

**Application:**

Place the bath board with handle crosswise in your bathtub. The rubber holders should rest against the edge, and the edge of the board should rest on the edge of the bathtub. Check to make sure the bath board is secure. Do not use it if it rests tightly on the edges of the bathtub or otherwise wobbles insecurely. If the bath board rests securely, the person using it can sit on it and wash and shower in the sitting position. The handle provides support when sitting down and standing up.

### Safety instructions:

1. The bathtub board is intended for domestic use in the bathroom and should be cleaned and dried after each use. The thermoplastic material is very resistant and not susceptible to corrosion. During use and cleaning, a temperature of 80 °C should not be exceeded.
2. Ensure that the bathtub board rests on both sides and at least 2 cm on the bathtub rim.
3. Before EVERY use, check that the handle and the rubber brackets are firmly fixed on the board.
4. Ensure that the bathtub board is firmly fixed on the bathtub rim before using it. This must not roll sideways!
5. Notice: The rubber holders must be firmly attached to the board, **but they must not be screwed too tight- the washers will otherwise be damaged and break!**  
The maximum load capacity of 150 kg must not be exceeded!
6. Do not use a broken bathtub board!
7. If you notice damage or changes to the material, contact your dealer.

### Cleaning

Clean the bathtub board regularly with a mild detergent and clear water. Use a soft cloth to dry. Do not use abrasive or corrosive cleaning agents. After each use, dry the product with a soft cloth in order to avoid build up of deposits. To disinfect the bathtub board, use a commercially available disinfectant. Do not use a high-pressure or steam cleaner for cleaning!

### Multiple use

The bathtub board is designed for daily use. The product can be used by several persons. Before the board is used by another person or passed into the possession of another person, the board must be thoroughly cleaned and disinfected in order to prevent the transmission of germs (contamination). This should be done as described under the point *cleaning*, and a commercially available disinfectant should be used.

### Storage

During use in the bathroom on the bathtub rim, good ventilation must be provided to prevent the mould formation on the bathtub board. Make sure the bathtub is dried before storing it, when it is not being used. Then place the product in the supplied plastic bag and box in a dry and dark place.

### Maintenance

The bathtub board with handle is maintenance free. If you notice damage or changes to the material, contact your dealer. Do not use a broken bathtub board!

### Contraindication

The use of the bathtub board is not suitable in case of paralysis of the upper part of the trunk and loss of upper limbs, as well as vestibular and cognitive disorders.

### Scope of delivery

Bathtub board, 4 rubber holders with 4 washers and 4 screws (plastic), 2 screws for the handle, plastic bag, carton, instructions for use

### Technical data/dimensions and weight for article 01840222

Material: Polypropylene (PP) = thermoplastic

Weight: 1.8 kg

Length: 70 cm

Width: 31 cm

Height: 5 cm

The adjustment range of the rubber holders (possible bathtub internal dimensions): 35 to 61 cm

Useful life: depends on the load and maintenance/care, but at least 5 years

Max. load capacity: 150 kg

**Technical data/dimensions and weight for article 09233208**

Material: Polypropylene (PP) = thermoplastic

Weight: 2.1 kg

Length: 75 cm

Width: 35 cm

Height: 5 cm

The adjustment range of the rubber holders (possible bathtub internal dimensions): 35 to 61 cm

Useful life: depends on the load and maintenance/care, but at least 5 years

Max. load capacity: 150 kg

**Notification/incident**

All serious incidents occurring in connection with the product must be reported to the manufacturer and the competent authority of the member state in which the user and/or patient is established.

**Warranty**

The warranty period is 24 months and shall apply to all product defects that can be attributed to manufacturing or material defects.

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Rehaforum Medical GmbH, Daimlerstraße 12a, DE-25337 Elmshorn