

carefix[®]

by Tytex

COMPRESSION LEVEL 2 - FIRM

POST-OP RECOVERY BRA FOR
POSTSURGICAL USE AFTER
BREAST SURGERY

AVA



ABOUT AVA ↴

- No irritating care label. Care instructions knitted in the bra
- Suitable when a firm level of compression is recommended.
- Front opening and openable shoulder straps for easy application when shoulder movement may be limited
- Available in 2 cup sizes
- Bilateral pockets for textile breast puffs



4-WAY STRETCH



SEAMLESS KNITTING



TARGETED COMPRESSION
AND FIXATION



HIGHLY BREATHABLE



FULLY OPENABLE

EUROPEAN-BASED PRODUCTION
FOR OPTIMAL QUALITY,
SUPPLY CHAIN EFFICIENCY
AND FLEXIBILITY



STANDARD
100
6234-7001
DTI

ARTICLE NUMBER: 3444

AVA

PRODUCT DESCRIPTION

Ava bra is designed for the time following a surgical procedure to the breast, when the skin is tender, sensitive and where support is recommended. The bra effectively fixate bandages and wound dressings worn in the post surgical phase. With a high back, it provides comprehensive protection and compression. The Ava bra support the users breasts during the recovery phase and ensuring optimal user comfort. A seamless design ensures no pressure is exerted on the wound and scar area. The bilateral pockets are perfect for textile breast puffs, allowing the user to create the ideal breast shape and achieve symmetry. With the front opening and openable shoulder straps, it is easy to apply the bra and it allows easy examination of the surgical site.

PRECAUTIONS

- Follow the Instructions for Use
- Follow your surgeon's recommendations regarding use and fitting
- If any unexpected discomfort or wound irritation occurs, remove the Post-Surgical Bra and contact your surgeon
- The Post-Surgical Bra is not intended for direct contact with injured skin or mucous membrane
- Make sure that bandages, textile breast prosthesis and/or drainage tubes are placed correctly to avoid pressure points and/or obstructed flow
- Before X-ray examination remove Post-Surgical Bra
- Wearing an undersized Post-Surgical Bra may cause local pressure pain and disruption of blood flow and/or lymphatic flow
- Wearing an oversized Post-Surgical Bra may reduce the clinical effect
- Use the Post-Surgical Bra according to medical indication to avoid injury or damage.
- The device may contain softening and antistatic agents up to 1.5 %.

**FEATURES**

- Front closure with zipper
- Adjustable shoulder straps with A-slide
- High back for rear thoracic support
- Pockets for textile breast puffs
- Tested for harmful substances according to OEKO-TEX STANDARD 100

INDICATIONS

For use 24 hour per day following; Breast augmentation, breast reduction, breast reconstruction, breast lift (mastopexy) and mastectomy.

RECOMMENDED SIZES

CM	65	70	75	80	85	90	95	100	105	110	115	120	125	130
IN	26	28	30	32	34	36	38	40	42	44	46	48	50	52
A														
B		S (CUP A-C)		M (CUP A-C)			L (CUP A-C)			XL (CUP A - C)				2XL (CUP A - C)
C														
D														
E		S (CUP D-F)		M (CUP D-F)			L (CUP D-F)			XL (CUP D-F)			2XL (CUP D-F)	
F														

* **NOTE:** Due to possible postsurgical swelling and edema, you may need to go up 1 - 2 sizes compared to your normal bra size. For more compression go down one size and for less compression go up one size.

Important: Always measure yourself to find the right size.

AVAILABLE COLOURS**COMPOSITION**

89% PA
11% EL

CARE HANDLING

Close the bra before washing
Wash with similar colours
Wash in a laundry bag
Use of chlorine based products may decrease durability and function